

3D PL UV Analyzer

Special Price 2x,xxx บาท



Combine All-in-One

(iScope + UV Scope + 3D Scope + Intelligent Software)

UV 3D Skin Analysis

『See what the skin experts see?』

Application

I Promote cosmetic products sales by effective proof

II Improve customer's satisfaction

----- For Laser / IPL /Thermage treatment results

III Help patients learn their skin status

IV New profitable market without great investment

UV 3D Skin Analysis

『Where could be used for it ?』

Range of use

Beauty salon
Beauty care center



Cosmetics distributor



Cosmetics stores



Cosmetics
manufacturers and
research institute



Hairdressing
health guidance



Nutrition
guidance counselor



Medicine dietary fang



Personal
health management




UV 3D Skin Analysis

『All-round Multi-function 』

Surface

Under Surface

UV

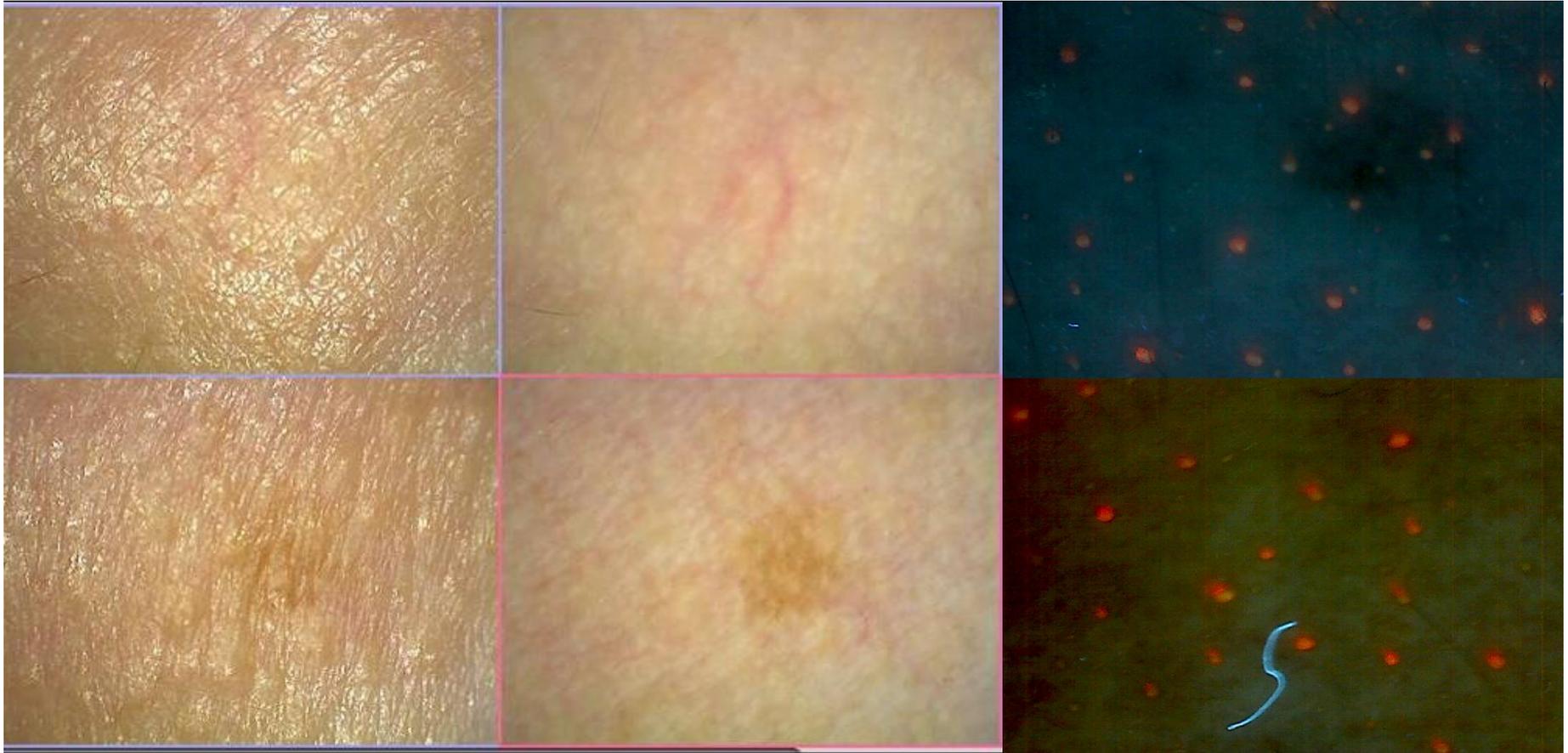

UV 3D Skin Analysis

『Advantages』

- Portable beautiful with handheld Design
- 5 million pixels with UV and 3D as the newest in skin analyzer
- Recommended Cosmetic products shop cart system is available
- Test 8 kinds of skin problem one time
 - moisture,oil,texture,complexion,3D,pigment,phlogosis,pores!
- Operating System: Compatible with Windows 8,Windows 7 and Vista
- No need the key-dog any more, save USB port
- CRM (Customer Relationship Management)System is available
- Analysis' results could be saved and printed

UV 3D Skin Analysis

『 Specification 』

- Imaging system: USA 1/3.25 inch CMOS;
- Hardware: 5 mega pixel;
- Maximum resolution: 1600*1200;
- Magnification factor: 50 times;
- Operating temperature: 10-40 °C;
- Operating humidity: below 80%;
- Power supply: USB 5V;
- USB 2.0 port, plug and play, no need driving

Software Screenshot

Intelligent for user

Whether make up or not



Analysis Creates Beauty

Company

Product

Login



Do you make up today?

Yes



No

4. Image Contrast

Multi Skin Analysis System




Image Contrast

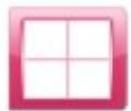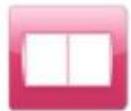

Customer Images



Standard Images

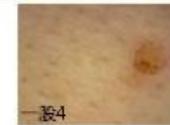


Multi Skin Analysis System



Image Analysis

Moisture	<input type="text"/>
Oil	<input type="text"/>
Texture	<input type="text"/>
Complexion	<input type="text"/>
3D	<input type="text"/>
Pigment	<input type="text"/>
Phlogosis	<input type="text"/>
Pores	<input type="text"/>

Surface Dermis Pores

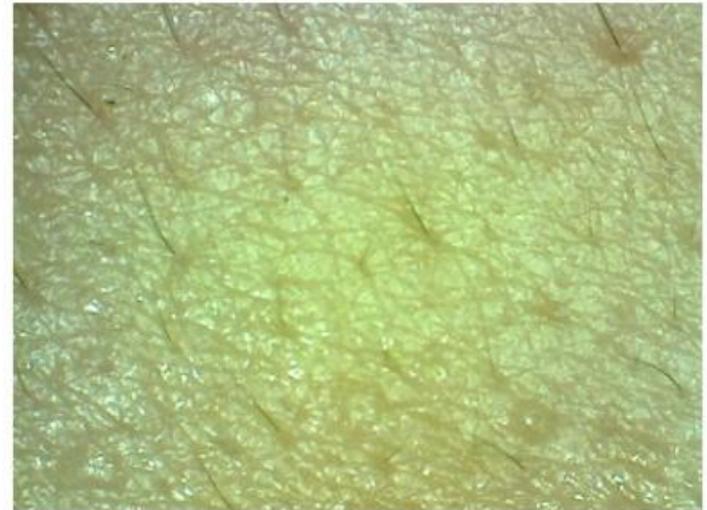



Multi Skin Analysis System



Image Analysis


Moisture	<input type="text"/>
Oil	<input type="text"/>
Texture	<input type="text"/>
Complexion	<input type="text"/>
3D	<input type="text"/>
Pigment	<input type="text"/>
Phlogosis	<input type="text"/>
Pores	<input type="text"/>

Surface Dermis Pores

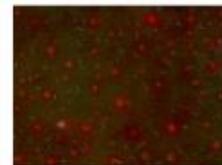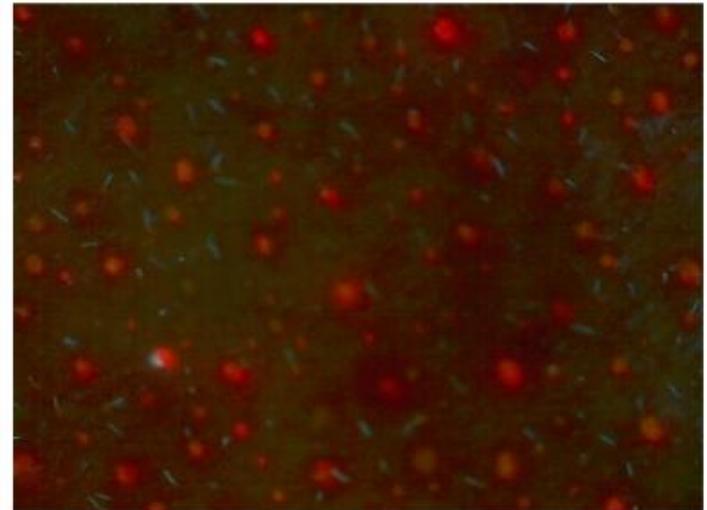



Result

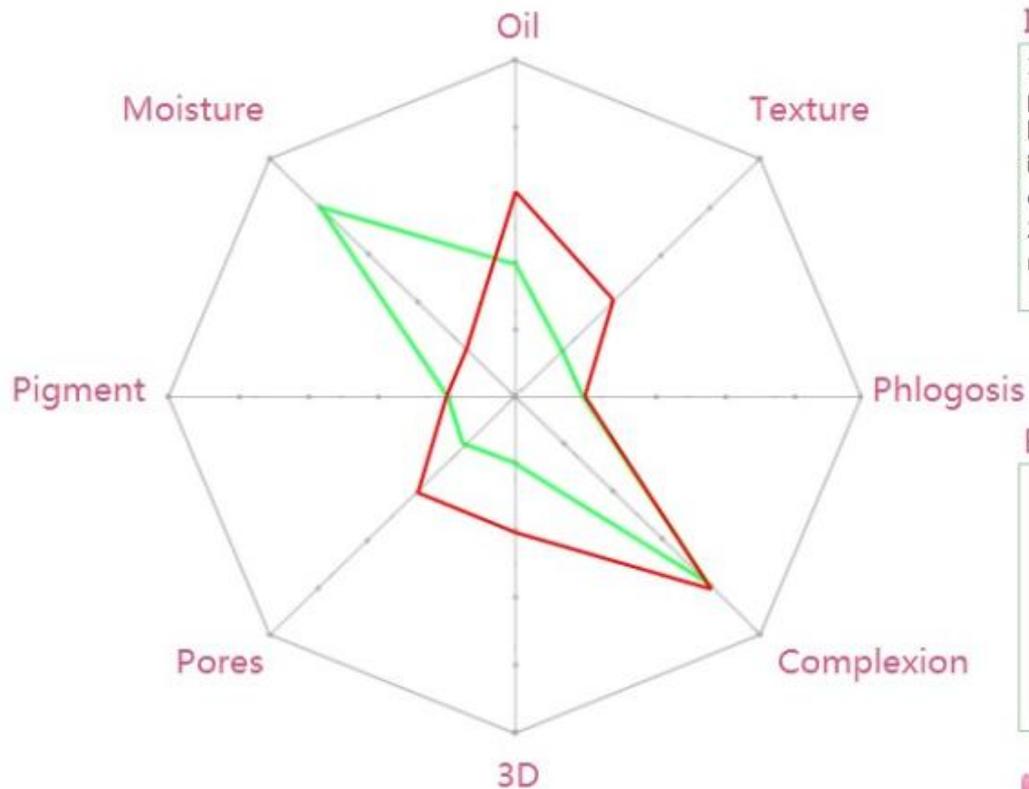

Integrated Proposal


- 1、 To increase skin nutrition and supplement vitamin A promptly;To maintain sufficient moisture in the organism and boost the metabolism of moisture in human body to get rid of impurity;To prohibit alcohol addiction and smoking;To be cautious of ultraviolet irradiation;To notice seasonal skin care.
- 2、 To forbid eat spicy and irritant food;To pay attention to maintain skin following the alternation of season.

Beautician Proposal

Empty text area for the beautician proposal.

[Product](#)[Print](#)[Return](#)


8.Result ---3D View

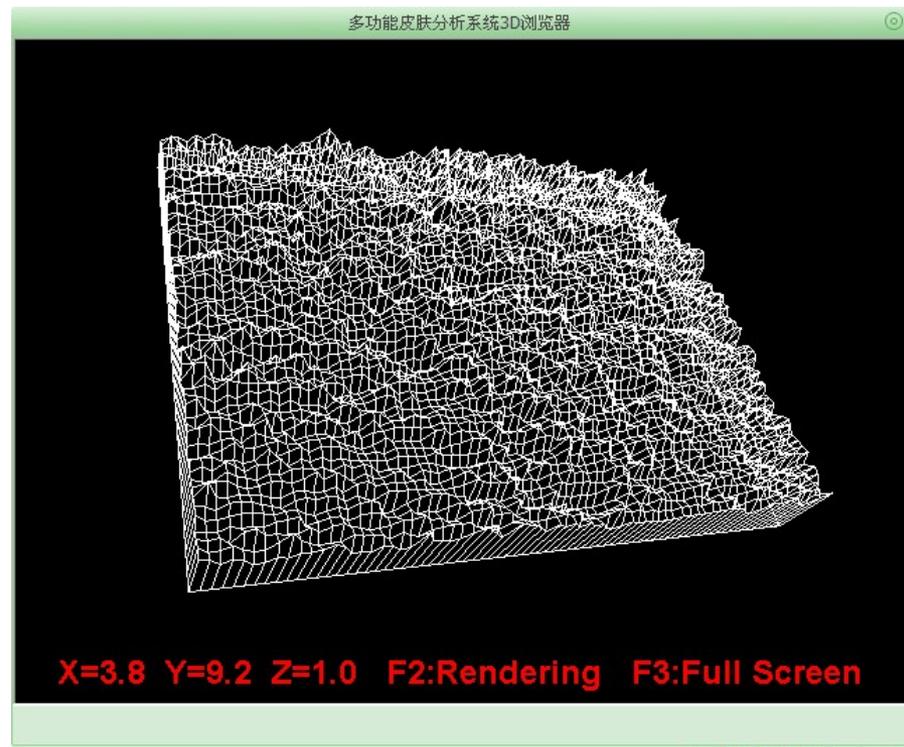



Complexion Test Results

Analysis Picture


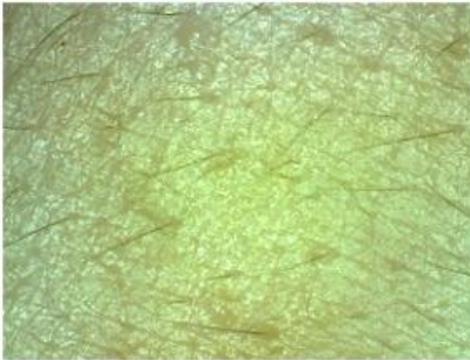


Advice

Pale skin color.


To promote blood circulation and enhance skin immunity.

Reference Picture

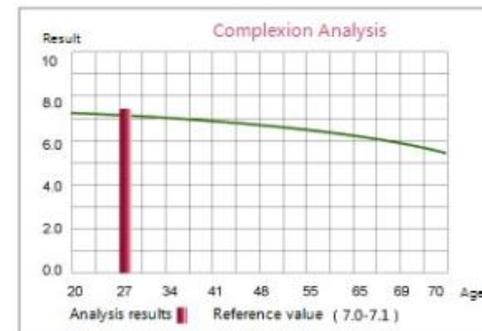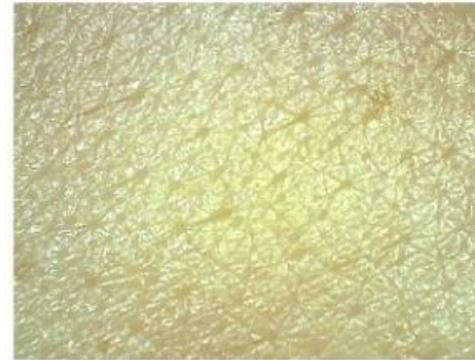

Next

Return

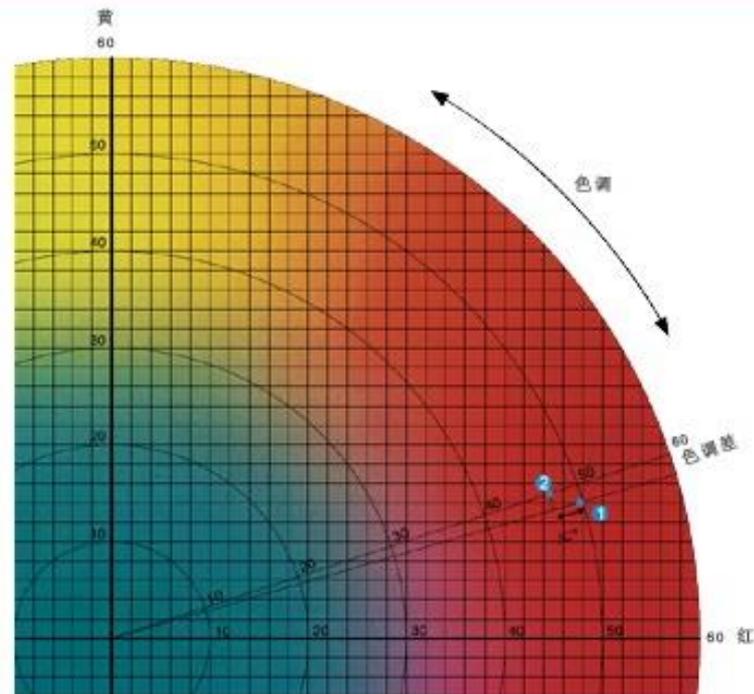

Complexion Analysis : Pale skin color.

Analysis principle

The skin color is decided by melanosome, hemoglobin, carotenoid, dermis blood vessels and dermis fiber, which will change obviously when they are impacted by ultraviolet, medicine and other irritant [Return](#)

The difference of the upside composition for stratum corneum is a vital factor to

11.Recommended Cosmetic Products Shop Cart

Multi Skin Analysis System



Moisture

Oil

Texture

Complexion

Pigment

Phlogosis

Integrated Proposal

Average moisture in bottom layer.
Excessive grease secretion.
Distinct but slightly irregular.
white touched with red.
Wrinkle gradually appearing.
Pigmentation.

Expert Proposal

To maintain regularly one or two moisturizing
nursing every week.
To make face cleansing and pay attention to
suitable moisturizing and sun protection.
To make anti-wrinkle nursing regularly.
To promote metabolism of skin to make it
more healthy and beautiful.

Product Details (large image)

Add

Cart

More Products

Return